Info: The most current version of these instructions can be found online at www.grahamfield.com.


- ⚠ **WARNING: Important! Read and understand these instructions before assembling or using your Lumex Drop Arm VersaMode. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If the VersaMode is not properly adjusted, personal injury and damage to the VersaMode could result.**
- ⚠ **WARNING: If components are damaged or missing, contact your GF authorized distributor immediately. DO NOT use substitute parts. Use only Lumex replacement parts. Non-Lumex replacement parts could cause personal injury and damage to the VersaMode.**
- ⚠ **WARNING: This product has a maximum weight capacity of 300 lb, EVENLY DISTRIBUTED.**
- ⚠ **WARNING: Notice for California Customers- California Proposition 65 WARNING: This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.**
- ⚠ **WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly or use of this product.**

INTENDED USE

To provide transport, transfer, and commode functions with one device.

ASSEMBLY (SEE PICTURE AT RIGHT)

1. Carefully remove components from box.
 2. Install back and arm assembly on base and seat frame assembly by inserting back tubes into base frame tubes. If necessary loosen, but do not remove, knobs.
 3. Tighten knobs at rear seat until secure — do not overtighten.
 4. Install footrests on base frame footrest brackets. Lock footrests into place.
 5. Slide the commode pail in under the seat from the back.
- ⚠ **WARNING: Before use, ensure all components are securely installed and VersaMode is assembled as described above.**

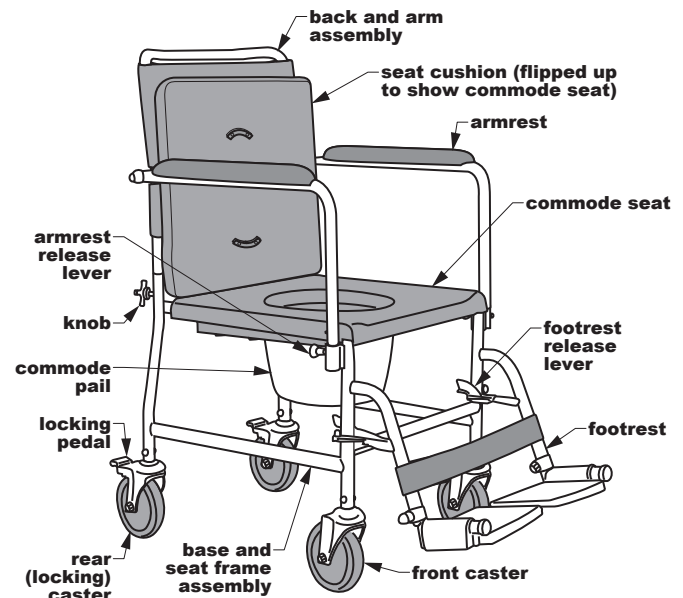6810A Drop Arm VersaMode

OPERATION

Drop Armrests

The VersaMode armrests drop to facilitate transfer. To drop the armrest, pull the release lever outward until the armrest releases, lift the armrest up out of the socket, then swing it downward.

Lock Rear Casters

The rear casters lock into place to prevent movement. To lock, step on the rear pedal until it latches. To unlock, step on the rear pedal until it unlatches.

Swing Away / Remove Footrests

The VersaMode footrests swing away or detach to facilitate transfer. To swing the footrest away, depress the footrest release lever and swing it out of the way. To remove the footrest, swing it away, then lift it up off the brackets. To lock the footrest in place, swing it in toward the center of the VersaMode until it locks into place.

MAINTENANCE

Check the fit and tightness of all nuts, bolts and knobs weekly.

⚠ **WARNING: Inspect the VersaMode weekly for proper function and wear. If any component is not in proper working order or appears worn, contact your GF authorized distributor for repair and appropriate replacement parts.**

Cleaning

Clean the VersaMode with a mild soap and water solution, rinse with clear water, then dry with a soft cloth. Ensure the VersaMode is well-rinsed before use.

▲ **NOTICE: Never use solvents, abrasive cleaners, or scouring pads on any part of the VersaMode.**

▲ **NOTICE: Always clean VersaMode as soon as possible after use. To contain odor until VersaMode can be cleaned, replace commode lid on commode pail immediately after use.**

LIMITED WARRANTY

The following warranty information is a SUMMARY of the complete Product Warranty provided by GF Health Products, Inc. For the complete warranty, please visit our website at www.grahamfield.com.

GF Health Products, Inc. ("GF") warrants to the original retail purchaser or lessee (either is the "Customer") only, that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. The warranty is for parts only and does not include labor or costs of shipping. All warranties are conditioned upon the proper use of the product strictly in accordance with applicable GF instructions. Product provided to users as rentals must be serviced by the provider and inspected prior to delivery. Each user must be properly trained on the operation and safety of this product prior to use. Within the guidelines set forth in this document, the frame is warranted for three (3) years; seat/back, extension legs, and arms are warranted for six (6) months; and the normal wear components such as tips, wheels/casters, and handgrips are warranted for three (3) months. The applicable warranty period shall commence from date of shipment to the Customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of the warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Dealer from whom the Customer purchased the product within the applicable warranty period. If there is not a Dealer, you must contact GF directly by calling 770-368-4700, sending a fax request to 770-368-2386 or by e-mailing a request to cs@grahamfield.com. Specific directions will be provided by the Customer Service Representative.

EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following: 1) Defects, damage, or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; 2) Products not properly installed, used, or cleaned and maintained; and 3) Labor or shipping charges. Other exclusions apply.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED.

For the complete warranty for this product, please visit our website at www.grahamfield.com.

GRAHAM-FIELD
www.grahamfield.com

GF Health Products, Inc. is not responsible for typographical errors. For the most updated and current information on packaging, warranties, products and specifications, including the most current version of these instructions, please visit our website at www.grahamfield.com.
Graham-Field and Lumex are registered trademarks, and VersaMode is a trademark, of GF Health Products, Inc.

© 2009 GF Health Products, Inc. • 2935 Northeast Parkway, Atlanta, GA 30360 • tel 770-368-4700 • fax 770-368-2386 • Made in China

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