

OTTO

QUICK START GUIDE

WELCOME



movinglife[™]

www.movinglife.com

Congratulations!

You are about to use the best
freedom mobility scooter
in the world!



We are at your service and will not
rest until you enjoy your Atto.
Feel free to contact us.

movinglife.com/quickguide


Unpacking flow:

- Hold the black handle of the ATTO and slide it out from the box.
- On the top left and right sides, there are two compartments (connected to the big box). Open them and you will find the battery in its box on the left one, and the charger & lubricant bottle on the right one.
- You will also see a clear plastic bag with the Owner's Manual, which contains all the relevant information about your ATTO, and a Flight Safety Certificates booklet (yellow) which contains relevant information required by some airlines for flying with the ATTO.

Now you have all parts at your side - ATTO + battery + charger + lubricant + plastic bag


Opening your ATTO:

1. Release the adjustment lever of the handlebar and bring it to an upright position, then bring the lever back to its secured position.

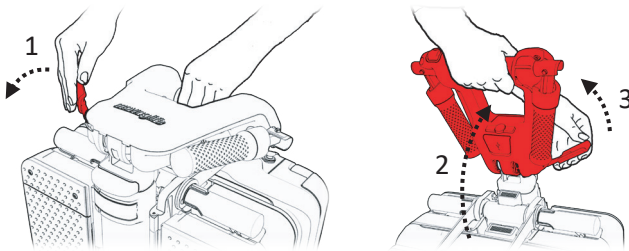


Now you are about to open the ATTO to a scooter mode - Are you excited?

2. Press the small red button, release the handle, and tilt it away from the main ATTO body - no force required - do it gently.

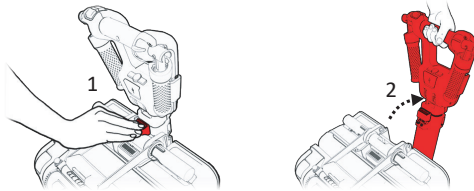


-
3. Press the big red button.

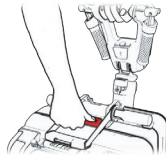

The following instructions are for right handed users (if you are left handed, please use the other hand than written).

4. Hold the handlebar with your left hand and the main handle which is horizontal on the ATTO body with your right hand.

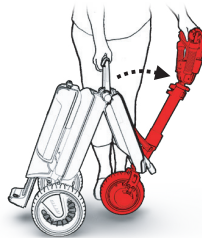


This part is a bit confusing in the beginning, be patient, after doing it several times - it will be much easier!

5. Using your right hand, tilt the ATTO towards the ground, while lifting the tiller with the front wheel a little from the ground with your left hand,

until you hear a click. It is suggested to put your right foot before the rear wheel of the ATTO so it won't roll forward when you do the step above.


6. When you hear the 'click' sound, lower your left hand and put the front wheel on the ground.
7. Let the back part of the ATTO roll backwards to the open position. Gravity will help in this stage. You can leave your right hand while still holding the tiller with your left hand.
8. Lock the tiller in its upright position. In order to do so, keep the tiller upright and change hands (now hold it with your right hand). With your

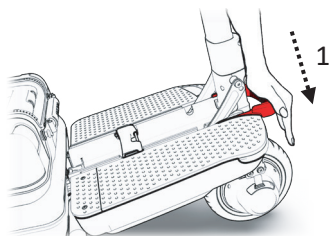


left hand push down the front handle (red in the illustration below) to lock it in position (make sure to do that when the tiller is vertical).

Opening the chair.


9. With your left hand hold the vertical tiller and with your right hand release the secure lever of the back handle (no. 1 in the illustration below). Lift and hold it in 90 degrees (don't go beyond the 90 degrees as this will lock the opening mechanism. If you hear a 'click' sound you opened too much). Now, with determination lift the back handle pulling the seat all the way to the back (away from the tiller) until it is locked.

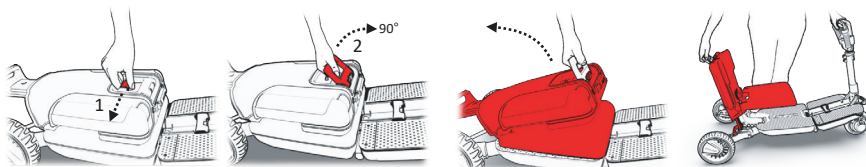

10. Take the upper rear lift handle and lift the chair up to a sitting position.

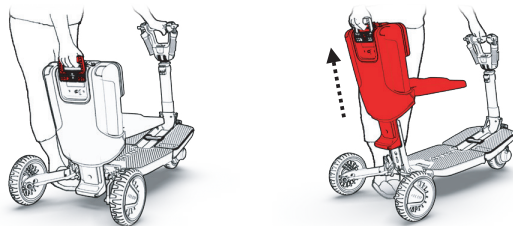


11. Take the battery out of its package and place it in the empty battery bay located in the left front side of the ATTO. First insert the back part of the battery and then slide it down giving it a slight push downwards until you hear a click.

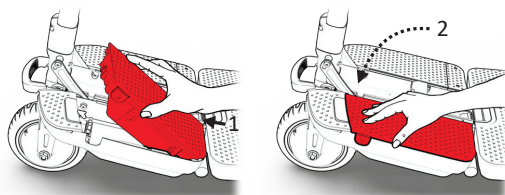


Time to sit on the ATTO's chair.


12. Put your hand on the middle black console and with your other hand open the lever (on the front bottom of the tiller) with your right hand lift the tiller up, then close the lever once adjusted.

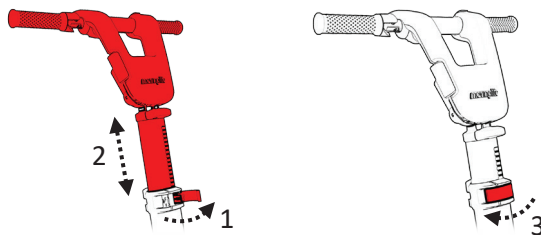

13. Open both hand grips by lifting them up.

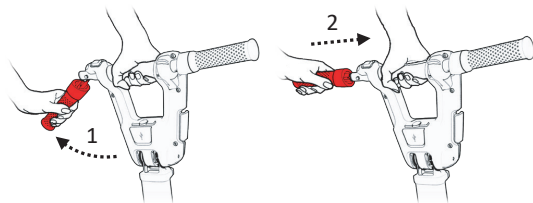


14. Now you can adjust the angle of the handle bar – Hold the console and open the side lever until the end. Adjust the angle to your comfort driving mode, then close back the lever.

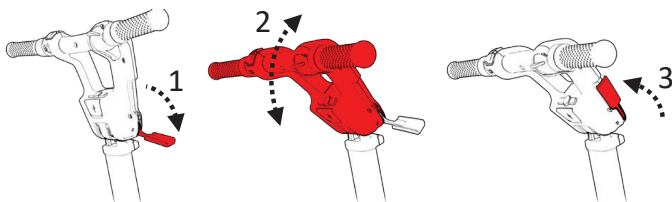


Your ATTO is ready to go!

15. Look at your ATTO's driving console. Press the main On/Off button (round button - see image on the left below) until the lights turn on. The number you see is the current speed limit set. You can adjust this setting by pressing the "+" or "-" buttons. We recommend starting at a low speed. On the upper side you will see two buttons. One is the horn and the other one is the direction button. If you press it, you can change the driving direction forward and backwards (you will see the direction with the arrow located on the right of the speed limit number).

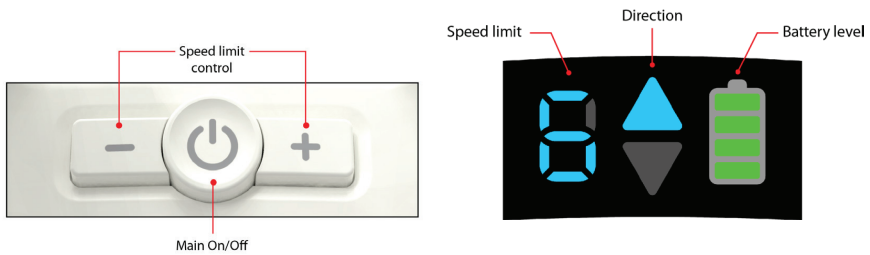


16. Use your right thumb to operate the drive throttle - if you press it downwards until the end, you will reach the speed limit that your ATTO is on (the number that appears on the board). You control your actual speed by the way you press the throttle.

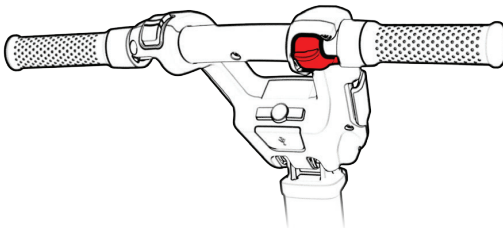


17. Stopping - just let go of the throttle and ATTO will automatically break.

Folding your ATTO:


18. First, turn off your ATTO. Push the main On/Off button and make sure the lights turn off.
19. Bring the handle bar back to its upright position: Open the lever on the right side of the handle bar, lift the console to the vertical position and then close the lever.
20. Fold the hand grips: Hold the center of the handle bar and pull the hand grip all the way out and slide it down. Do the same on the second hand grip as well.

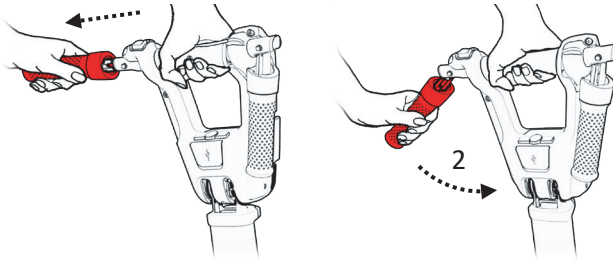


-
21. Hold the console, on the bottom of the tiller open the lever, push the tiller gently down. close the lever.

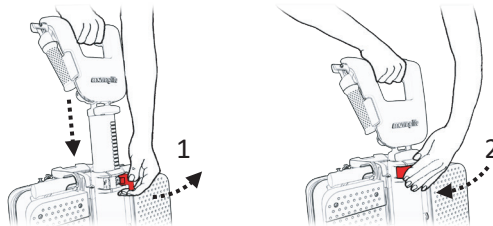

22. It's time to stand behind the chair, release the secure lever while bringing the handle to 90 degrees, easily, while holding, push the chair down until the end. Place your hand on the back of the chair and push it all the way inwards

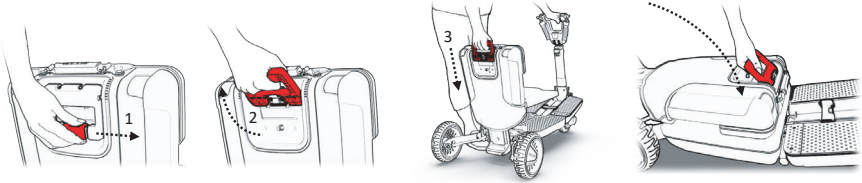

23. Use one hand to hold the rod upright, with your other hand release the front lever by pulling it upwards (look for the pinch point and be careful). After releasing, change hands, with your free hand lift the main handle up.

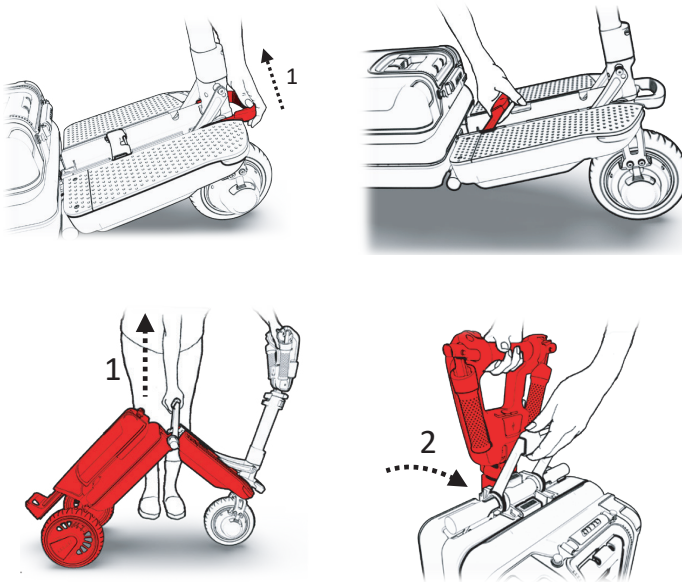



movinglife.com/quickguide

In our full Owner's Manual or website
you can find more information about:
separating your ATTO into two parts,
traveling, flights, safety and much more.

We wish you a safe and happy ride!!!

This Quick Guide does not replace ATTO's Owner's Manual.
Please read it carefully before operating your ATTO.

movinglife[™]

We are here
For you



movinglife.com/quickguide

*moving*life™