

Quick Start Guide

GP605 – Compass Sport



Read before assembly.

Step 1.

Remove the front and rear battery covers by lifting as shown. Make sure both battery harnesses are plugged into the power chair and locked. Make sure the power harness is plugged into the Dynamic power module and locked. Install the front and rear battery covers. Make sure the freewheel levers are in the engaged position. Unwind the joystick control cable from around the seat post. See figure 1 and figure 2.

Step 2.

Hold the seat at an angle to visually align the seat swivel post, and the seat post on the power chair. Lower the seat swivel post into the seat post. Push down to make sure it is seated securely and unable to move side to side. Push down on the swivel handle and move the seat so that it is facing in the forward position if not already there. See figure 3.

Step 3.

Insert each arm into the arm receiver tube at the bottom rear of the seat frame. Adjust the width of each arm. Install and tighten each arm width adjustment knob to secure each arm in place. See figure 4.

Step 4.

Loosen the joystick bracket adjustment knob located under the armrest. Insert the joystick into the bracket under the armrest. Adjust the position of the joystick to a comfortable spot. Tighten the joystick adjustment knob to secure the joystick. Route the joystick control cable inside the arm receiver tube and the arm. Connect the cable to the joystick. Secure the joystick cable to the receiver tube, and to the joystick bracket clip. See figure 5.

Step 5.

Sit on the seat, facing forward, and push the on/off button on the joystick. Verify the battery meter on the joystick illuminates indicating power on and that the status indicator is not flashing. If it is flashing, count the number of flashes, and refer to your owner's manual to determine the problem. If the battery meter is illuminated and the status indicator is not flashing, your power chair is assembled correctly, and you are ready to ride.

NOTICE

New batteries must be fully charged prior to the initial use of the product. Connect the charger to the power chair first, then the wall outlet, and charge the batteries for 12 hours before using the product.

Step 6.

(Optional) Connect the accessory bracket to the arm receiver tube. Only use the supplied hardware and mount through the predrilled holes located at the rear of the seat. See figure 4.

